

How to Transport the Compella Bariatric Bed



KIDD elevators must be used to transport this bed

PREPARING BED FOR TRANSPORT

- Raise siderails into locked position
- Press and hold **“Width Retract”** control until bed is fully retracted
- Press and hold **“FlexAfoot”** control to adjust length of bed, if necessary
- Unplug power cords and store on blue cord wraps

ENGAGING TRANSPORT MODE

- On the Transport Pod, press green **Transport** control to lower the bed into transport position
 - A single audible confirmation beep will sound and green indicator light will illuminate to confirm the bed has reached transport mode
- Set the green brake pedal to **Steer**
 - A green indicator light will illuminate once brake pedal has been set to steer
- To move bed, grip blue transport handles and depress green enable switches

AFTER TRANSPORT

- Disengage transport mode by pressing **“Disengage”** control to raise bed out of transport position
- Position bed and then, set the orange brake pedal to **“Brake”**
- Plug power cords into AC power
- Press and hold **“Width Expand”** control until bed is fully expanded
- Press and hold **“FlexAfoot”** control to adjust length of bed, if necessary