Health o meter[®]

P r o f e s s i o n a l

Model 500KL

Eye Level Digital Beam Scale

**ISO13485:2003
CERTIFIED**

E112697

UL 60601-1

User Instructions

PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA
© Pelstar, LLC 2016

www.homscales.com

P/N UM500KL Rev 20160817-Web

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

TABLE OF CONTENTS

Cautions and Warnings	3
Specifications / Definition of Symbols.....	5
Certifications / Connectivity / Disposal	6
500KL Assembly Instructions	7
Set up / Batteries.....	14
Operating Instructions	16
Maintenance.....	21
Calibration	22
Troubleshooting	23
Warranty.....	24

Note: This scale has been factory calibrated, and does not require calibration prior to use.

Please register your scale for warranty coverage at:

www.hom scales.com

For User Instructions updates and revisions please go to:

www.hom scales.com

CAUTIONS AND WARNINGS

When operating this scale:

- To prevent injury and damage to your scale, please follow the enclosed user instructions to assemble and operate very carefully.
- Exercise caution when assembling the scale column so as to not pinch the load cell connector cable.
- For accurate weighing, this scale must be placed on a flat, level and stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Operating this scale at voltages and frequencies other than specified can damage the equipment and will void the warranty.
- Do not transport the scale with any weight on the platform.
- Do not exceed recommended weight limit of 500 lb / 220 kg for this scale.
- Do not use in the presence of flammable or explosive materials.
- For accurate weighing, if the “Lo” indicator activates, replace the batteries or switch to the optional AC adapter as soon as possible.
- If the scale becomes damaged, it should not be operated until properly serviced.
- When replacing / inserting batteries, be sure to use all new batteries.
- To prevent patient injury, the patient must be attended throughout the entire weighing event.
- This equipment has been tested and found to comply with the EMC limits for the Medical Device Directive 93/42/EEC (EN 55011 Class A and EN 60601-1-2). These limits are designed to provide reasonable protection against harmful interference in a typical medical installation. The equipment generates, uses and may radiate radio frequency energy and, if not installed and used in accordance with these instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference with other devices, which can be determined by turning the equipment off and on, the user is encouraged to try to eliminate the interference by one or more of the following measures:
 - Reorient or relocate the receiving device.
 - Increase the separation between the equipment.
 - Connect the equipment into an outlet on a circuit different from that to which the other device(s) is connected.
 - Consult Customer Service or a field service technician for help.

CAUTIONS AND WARNINGS

When operating this scale with the AC adapter:

- Operate this scale exclusively with the AC adapter provided by Health o meter Professional. Use of an unspecified adapter will void the warranty, and can pose a serious safety hazard.
- For use with AC adapter model ADPT31 (Domestic, USA) or ADPT30WA (International, plug kit)
- Prior to using this scale, inspect the AC adapter cord for cracking / fraying, or for broken / bent plug prongs.
- Prior to using this scale, plug the AC adapter into the scale first, then into the wall outlet.
- Prior to using this scale, ensure that the AC adapter is plugged into an outlet with the rated voltage appropriate for operation.
- Ensure that the AC adapter outlet is wired to a circuit breaker or other protected power source.
- Ensure that the AC adapter and scale do not come in contact with liquids, excessive temperature, or excessive humidity.
- Unplug the AC adapter, and carefully store both the adapter and adapter cord before moving the scale.
- If the scale is being connected to another medical device, the power adapter must not be used. Disconnect the power adapter from the scale and use only the battery power.

SPECIFICATIONS FOR THIS SCALE

General

The Model 500KL eye level digital beam scale uses highly sophisticated microprocessor technology. Each precision scale is designed to provide accurate, reliable and repeatable weight measurements. In addition, each scale is designed to provide the user with features that make the weighing process simple, fast, and convenient.

Weight can be displayed in pounds (lb) or in kilograms (kg). **This scale can be powered using an AC adapter, model ADPT31 (not included), or by 6 - AA batteries (included).**

500KL Scale Specifications

Capacity and Resolution	500 lb x 0.2 lb (220 kg x 0.1 kg)	
Power Requirements	120 - 240AC, 50 - 60Hz - 9V DC adapter or 6 - AA batteries	
Environmental	Operating temperatures: 50°F to 104°F (10°C to 40°C) Storage temperatures: 30°F to 122°F (0°C to 50°C) Maximum Humidity: 95% RH	
Physical Dimensions	Scale (Overall Dimensions) Width: 14-1/8" (36 cm) Depth: 21-1/8" (53.8 cm) Height: 53-5/8" (136.2 cm) Weight: 21 lb (9.5 kg)	Platform (Standing Area) Width: 13-3/4" (34.9 cm) Depth: 16-1/2" (41.9 cm) Height: 2-3/8" (6.0 cm)

DEFINITION OF SYMBOLS

Attention, Consult Accompanying Documents

Temperature range


AC to DC power supply

Type BF Applied Parts


Humidity range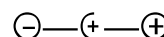

DC Connector Polarity

AC to DC power supply

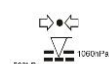Atmospheric pressure range

CERTIFICATIONS / CONNECTIVITY / DISPOSAL

Certification Descriptions

	Pelstar, LLC has been officially certified as a manufacturer of Health o meter[®] Professional medical devices to be in compliance with stringent ISO requirements. The Pelstar, LLC quality assurance system covers the design, marketing, and distribution of specialized scales for the Healthcare Industry.
	This product is tested to and meets the requirements of: 1. UL60601-1 First Edition: Safety for Medical Electrical Equipment 2. CSA C22.2 No. 601-1 M90: Medical Electrical Equipment

500KL Scale Connectivity Information

To enable reliable transmission of weight, height, and Body Mass Index (BMI) data, this scale is designed to connect to a computer, monitor, or other electronic device via a built-in USB port. This connectivity helps minimize a source of potential medical errors caused by incorrectly copying and then recording patient data.

This scale is EMR-EHR compatible. For more information regarding how to connect this scale to other electronic devices, please contact Health o meter[®] Professional Scales Technical Support, available during normal business hours at 1-800-638-3722.

500KL Scale Disposal

This Health o meter[®] Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal of electronic waste or batteries. Do not dispose of this device in the domestic waste stream.

500KL ASSEMBLY INSTRUCTIONS

Before Assembly

Each 500KL eye level digital beam scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter® Professional representative immediately at 1-800-815-6615. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the parts for assembly.

To prevent scratching any components, carefully remove each assembly from the carton and unwrap the packing materials. Set the carton aside for storage. To avoid damaging the scale parts when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging.

Parts List

Carton

- (1) Scale platform - C
- (1) Upper column (w / display assembly) - A
- (1) Lower column - B
- (6) AA batteries
- (1) User instructions
- (1) Upper height rod (w / head assembly) - D
- (1) Lower height rod - E
- (4) Column mounting screws (attached to lower column B)


Tools Required

Phillips head screwdriver (not included)

500KL ASSEMBLY INSTRUCTIONS (CONTINUED)


Assembling the 500KL Scale

1. Carefully remove the contents of the carton and place each part on a flat, level, and dry surface.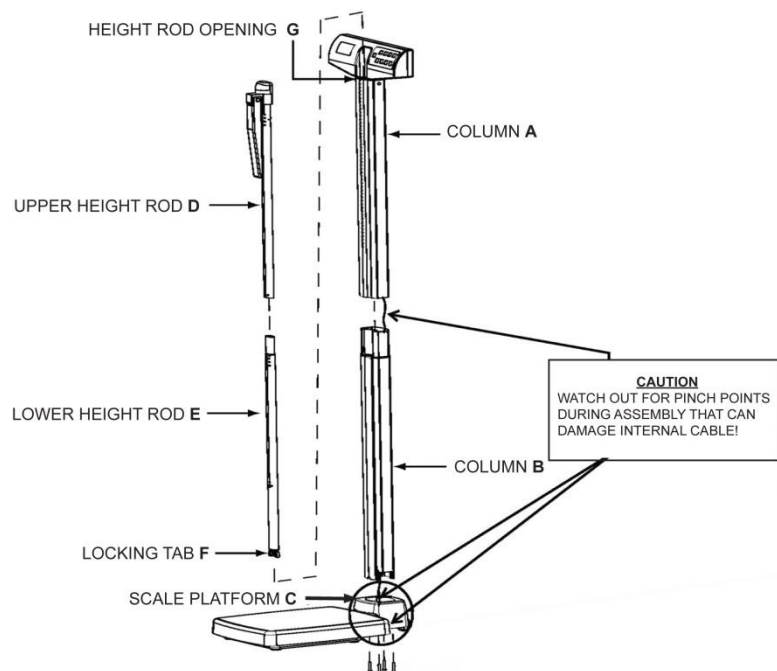


500KL ASSEMBLY INSTRUCTIONS (CONTINUED)

2. Remove the 4 column mounting screws and washers from the bottom of column B and set aside for assembly step 8.

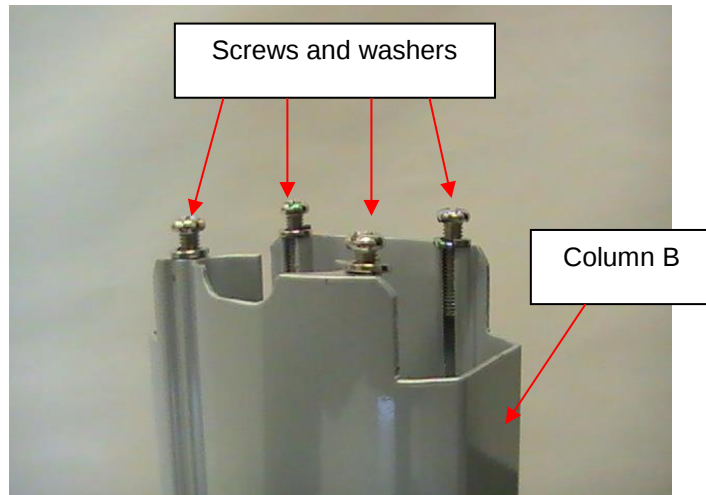
3. Remove the black bracket from scale platform C. Read the "Attention" label then remove it from the bracket. Place the bracket on the bottom of Column B with the adapter jack arrow sticker pointing towards the bottom of the column.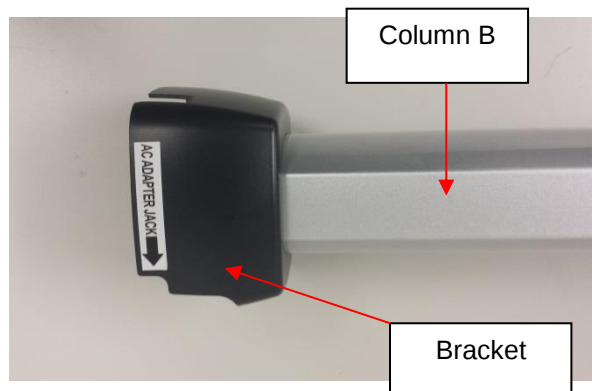


500KL ASSEMBLY INSTRUCTIONS (CONTINUED)

4. Locate the load cell connector cable inside column A. While holding column B, slide the cable through the center opening at the top of column B. Push the cable all the way through Column B until the plug reaches the bottom of the column.

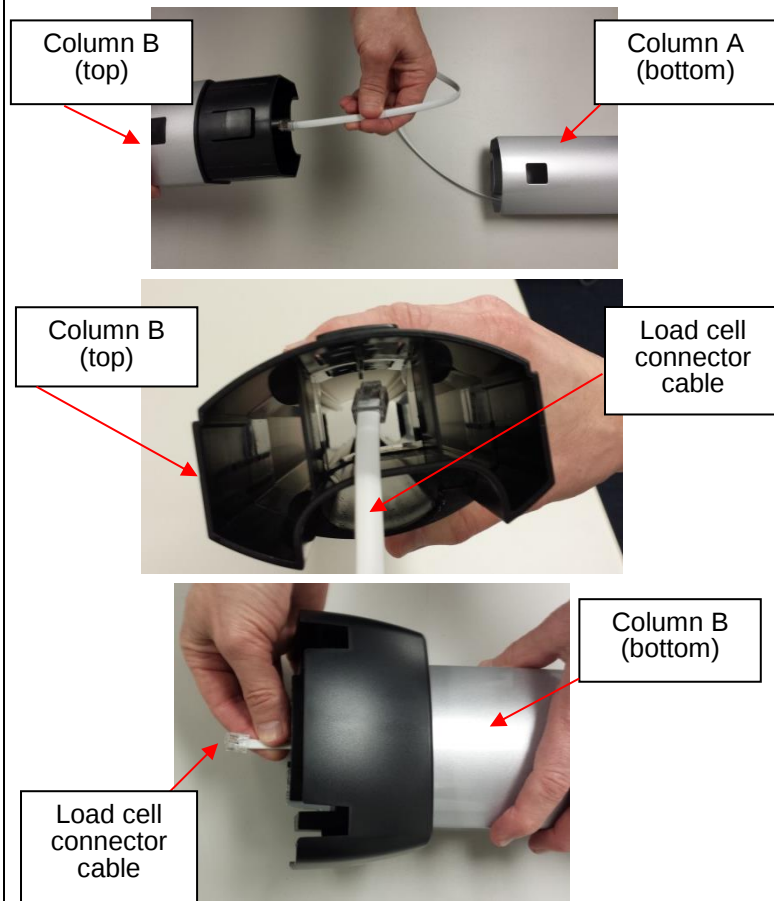
5. Align column A and column B. Notches are located at the bottom of column A and the top of column B. Snap the locking tab into place. Ensure that the locking tab is locked securely into place. **Note: When aligning and locking the columns, use caution not to pinch the load cell connector cable.**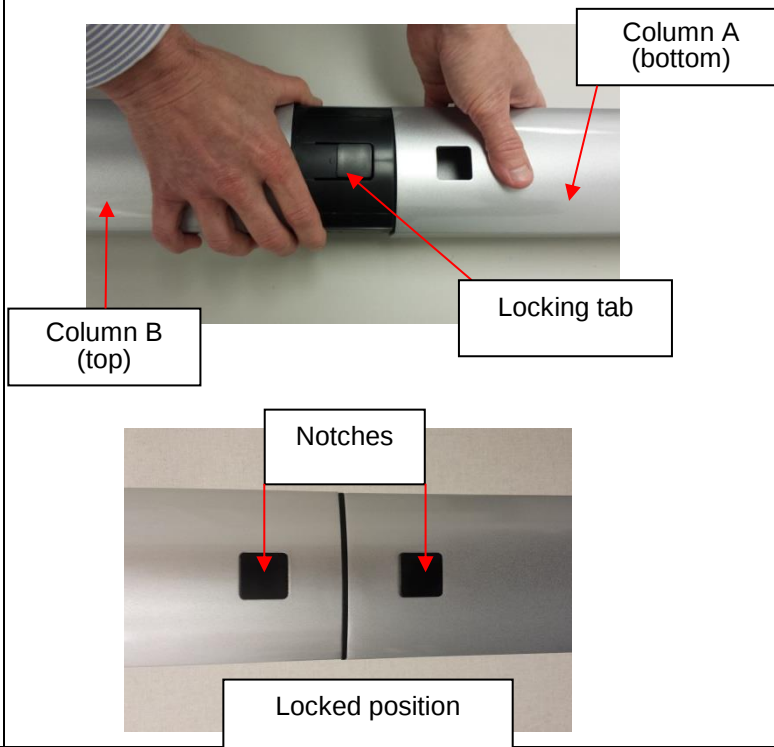


500KL ASSEMBLY INSTRUCTIONS (CONTINUED)

6. Place scale platform C on its side and place Column B on its side, perpendicular to the platform. The front of the display head on Column A should be facing forward and above scale platform C.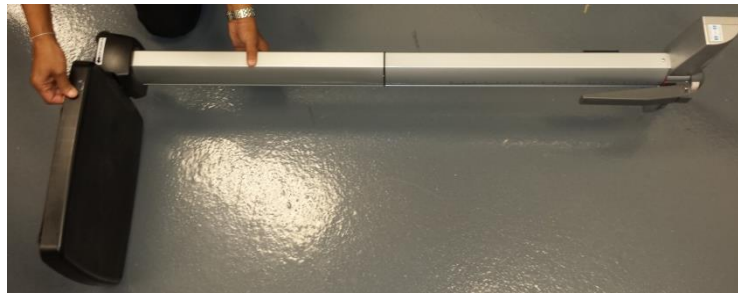

7. Carefully pull the cable connector from the bottom of Column B and plug into the RJ jack on scale platform C. **Caution: Once connected, keep a firm grip on the columns to ensure the load cell connector cable is not pulled out from the RJ jack. This cable connects the load cells to the display head and must be entirely plugged in for the scale to function properly.**

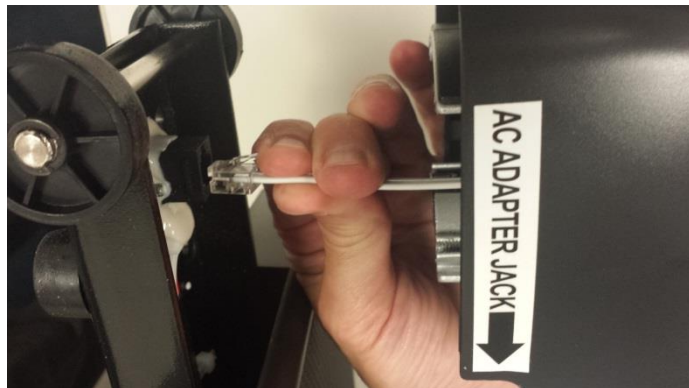
500KL ASSEMBLY INSTRUCTIONS (CONTINUED)

8. Position and secure the connected columns A and B to scale platform C. Locate the 4 column mounting screws previously removed in step 2. Insert the 4 screws through the bottom of scale platform C screw holes and into Column B. Ensure that washers are located between the screw heads and the scale platform C screw holes. Using a Phillips head screwdriver, tighten the screws to secure Column B to the base of platform C.


9. Height Rod Assembly: Locate the upper height rod D and lower height rod E. Align upper height rod D with lower height rod E. Snap the locking tab into place. Ensure locking tab is securely locked.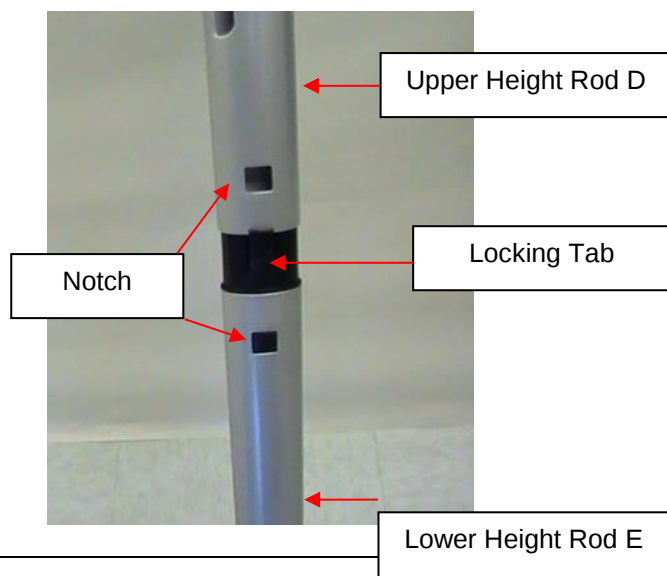

500KL ASSEMBLY INSTRUCTIONS (CONTINUED)


10. Position the bottom of the fully assembled height rod into the height rod opening G at the top of column A. Press locking tab F inward to insert the height rod into the height rod opening G. The scale is now assembled and ready to use.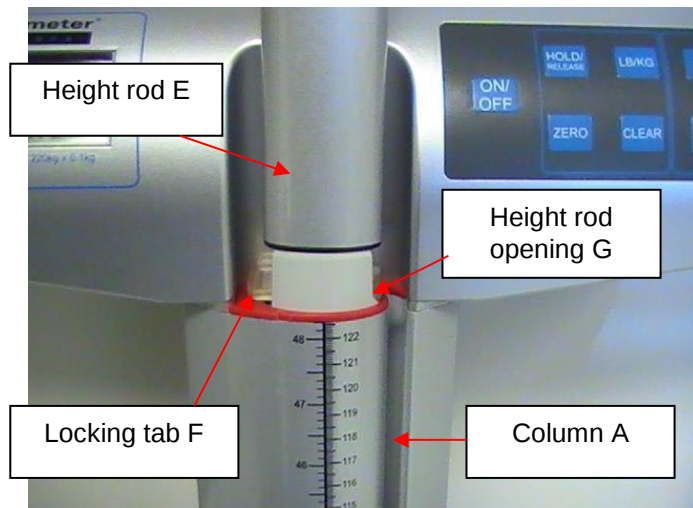

SET UP / BATTERIES

Preparing the 500KL Scale for Use / Inserting Batteries

1. Remove the protective plastic film covering the display (the film says "FACTORY CALIBRATED").
2. Unplug the optional AC adapter, if connected to a power source.
3. With a Phillips head screwdriver remove the screw from the battery compartment cover located on the back of the display. Remove the battery cover by pressing downward on the locking tab.

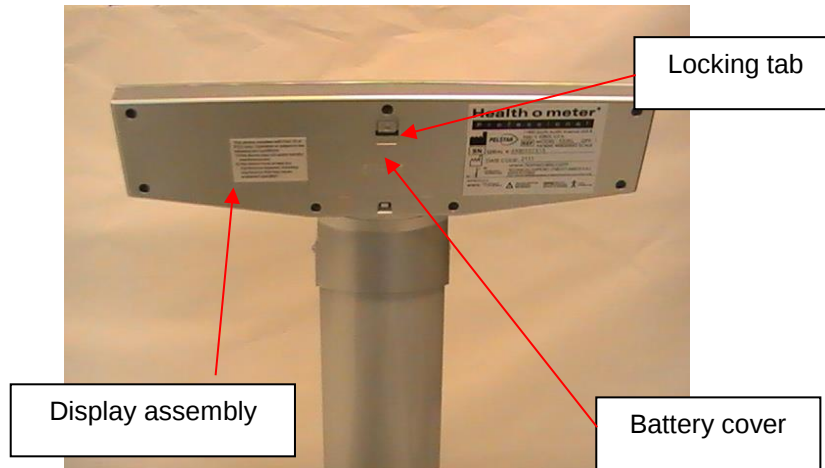4. Replace or install 6 new AA batteries. Health o meter® Professional recommends using an alkaline or lithium battery, rated for 130°F (54°C) operation. Do not use zinc-carbon batteries.
5. Re-attach the battery cover to the display assembly.
6. Ensure all objects are removed from the scale's platform, and then press the **ON/OFF** button to turn the scale on. The display will show "Start", then "0.0" LB. LB (Pounds) is the default weight measuring unit upon initial set-up/assembly. Press the **LB/KG** button to select your desired weight mode (LB or KG).

Note: The battery icon on the display does not indicate battery life. The battery icon is displayed when the scale is using batteries as the power source.

Note: This scale will always default to the settings and units last used (LB or KG).

Note: If the set up procedure fails, refer to the troubleshooting instructions. If the problem is not corrected, contact Health o meter Professional Scales Customer Service at 1-800-815-6615.

Note: If "Lo" appears on the display, it indicates that the battery power is low. It does NOT indicate a problem with the weight measurement.

Warning: If the scale will not be used for some time, remove the batteries to avoid a safety hazard. Disposal of batteries must be performed in accordance with the prevailing national, regional or local regulations which apply to you.

SET UP / BATTERIES (CONTINUED)

Optional AC Adapter

1. Connect the AC adapter (not included) into the adapter jack on the scale platform C.
Warning: To avoid a safety hazard, use AC adapter model ADPT31 (Domestic, USA) or ADPT30WA (International, plug kit)

2. Plug the scale's AC adapter into the power source.
3. When the AC adapter is disconnected from the scale, the scale will switch to battery power if batteries are installed.

Note: If the set up procedure fails, refer to the troubleshooting instructions. If the problem is not corrected, refer to qualified service personnel.

Caution: Always plug the AC adapter into the scale first, then into the power source/wall outlet.

OPERATING INSTRUCTIONS

Keypad

Function	Description
ON/OFF	Turns scale on and off.
HOLD/RELEASE	Holds the weight value of the patient on the display until the button is pressed again to clear the value.
LB/KG	Toggles between pounds (LB) and kilograms (KG).
BMI	Prompts entry of height data to calculate patient's Body Mass Index (BMI).
UP ▲	Increases height input (0.5 in / 0.5 cm increments)
ZERO	Zeroes out or tares the scale prior to a weighing.
CLEAR	Clears the incorrect height input when in BMI mode.
ENTER	Used to accept height value in BMI mode.
DOWN ▼	Decreases height input (0.5 in / 0.5 cm increments)

OPERATING INSTRUCTIONS (CONT)

Weighing a Patient

Note: When this scale is turned on, it will always default to the settings and units last used (LB or KG).

1. Make sure there is nothing on the scale platform.
2. Press the **ON/OFF** button to turn the display assembly on. “StArt” will briefly appear on the display, followed by “0.0” LB or “0.0” KG.

3. Ask the patient to step on the scale. The scale will begin to display weight information and will lock on the patient's weight when the stable weight of the patient is determined. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight.

Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.

4. The weight may be held on the display by pressing the **HOLD/RELEASE** button. To release, press the **HOLD/RELEASE** button again.
5. After the weight measurement has been taken and is recorded, have the patient step off the scale.
6. Press the **ON/OFF** button to turn the scale off.

Height Measurement Instructions

WARNINGS: Use caution when sliding height rod up and down. The rod should move smoothly; do not move the rod in a jerking motion. Fold headpiece down when not in use.

Note: Patients must be a minimum of 23-5/8" (600mm) tall for the height to be accurately measured.

1. With the headpiece in the folded down position, slide the headpiece up or down to the estimated height of the patient. Then raise the headpiece into its horizontal position.


OPERATING INSTRUCTIONS (CONT)

2. Position the patient facing the display head. Ensure that the patient's posture is upright, and that the patient's head is straight and level. Slide the headpiece down until it rests on the patient's head.

3. Determine the patient's height by reading the value on the measuring strip nearest to the line on the indicator, as shown below.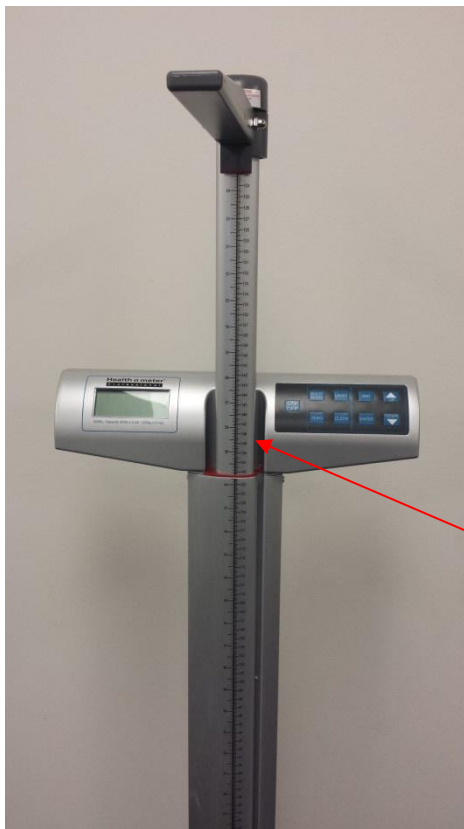

Read measurement here
for heights greater than
48.5" / 123 cm

Read measurement here
for heights less than
48.5" / 123 cm

OPERATING INSTRUCTIONS (CONT)

BMI Function

Body Mass Index (BMI) is a ratio between a person's weight and height and interpreted as a BMI score. This BMI score is then used as a tool to categorize a patient as underweight, normal, overweight, obese or morbidly obese.

Note: The scale will not calculate BMI for a patient that is 24 pounds (12 kg) or less.

Note: If a height is not entered for the patient within 30 seconds of pressing the **BMI** button, the scale will return to the normal weighing mode.

1. Make sure there is no object on the weighing platform.
 2. Press the **ON/OFF** button to turn the scale on.
 3. Wait until the "0.0" LB or "0.0" KG appears on the display.
 4. Ask the patient to step on the scale. The scale will begin to display weight information and will lock on the patient's weight when the stable weight of the patient is determined. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight.
 5. After obtaining the patient's weight on the scale, press the **BMI** button while the scale is on.
 6. The display will show 65" (in LB mode) or 150cm (in KG mode). Use the keypad **▲** and **▼** buttons to enter the patient's height. To determine the patient's height, follow the height measuring instructions on pages 17 and 18.
- Note:** When weighing in pounds, the height is displayed in inches. When weighing in kilograms, the height is displayed in centimeters.
7. Once height is keyed in, press the **ENTER** button.
 8. The display will show the patient's BMI.
 9. Press the **CLEAR** button to return to the normal weighing mode of the scale.

Auto-Off Function

As an energy saving feature, the scale will automatically turn off after 2 minutes of inactivity. By default, this function is enabled.

Disable Auto-Off

1. While the scale is turned off, press and hold the **HOLD/RELEASE** button.
2. While holding the **HOLD/RELEASE** button, press the **ON/OFF** button to turn the scale on.
3. "START" will briefly appear on the display followed by "AOF=0".
4. Auto-off is now disabled and the scale will remain on until the scale is manually turned off.

Enable Auto-Off

1. While the scale is turned off, press and hold the **HOLD/RELEASE** button.
2. While holding the **HOLD/RELEASE** button, press the **ON/OFF** button to turn the scale on.
3. "START" will briefly appear on the display followed by "AOF=1".
4. Auto-off is now enabled.

To Change Weight Measuring Unit (LB to KG/KG to LB):

Note: LB (Pounds) is the default weight measuring unit upon initial set-up/assembly.

1. Press the **ON/OFF** button to turn the scale on.
2. Wait for "0.0" LB or "0.0" KG to appear on the display.
3. Press the **LB/KG** button until the desired unit of measure is displayed.
4. Release the **LB/KG** button and the scale will operate in the selected mode.
5. Repeat steps 3-4 to change the weighing mode to a different unit of measure.

OPERATING INSTRUCTIONS (CONT)

LB/KG Lock Out

The weight measuring unit can be locked to only measure in KG or LB.

KG Lock Instructions

1. Press the **ON/OFF** button to turn the scale on. Wait for “0.0” to appear on the display.
2. Press and hold the **LB/KG** button so that KG weight measuring unit is displayed and continue holding the **LB/KG** button for approximately 10 seconds.
3. When the KG weight measuring unit flashes, release the **LB/KG** button. The KG weight measuring unit is now locked and will remain locked even if the scale is turned off.

LB Lock Instructions

Note: To lock in LB weight mode, you must follow the instructions above to lock in KG weight mode first.

1. Once the scale is locked in KG mode, press and hold the **LB/KG** button for approximately 10 seconds until the LB weight measuring unit flashes.
2. Once the LB weight measuring unit flashes, release the **LB/KG** button. The LB weight measuring unit is now locked and will remain locked even if the scale is turned off.

Disabling LB/KG Lock Out

Note: If your scale is locked in KG mode, you must first follow the instructions above to lock into LB mode, and then begin the disabling instructions below.

1. Press the **ON/OFF** button to turn the scale on. Wait for “0.0” to appear on the display.
2. Press and hold the **LB/KG** button until LB and KG *both* flash on the display. When both LB and KG are flashing on the display, release the **LB/KG** button. The weight measuring unit is now unlocked and pressing the **LB/KG** button will allow user to toggle between LB and KG.

Zero Function

Zeroes the scale prior to a weight measurement. The **ZERO** button is also used in the Tare function as outlined below.

Tare Function

When using this scale, the weight of an object, such as a cane, can be subtracted from the total weight in order to determine the weight of the patient alone.

1. With all weight off the scale, press the **ON/OFF** button to turn on the scale.
2. Wait until the “0.0” LB or “0.0” KG appears on the display.
3. Place the object on the scale. The display will show a value for the object’s weight.
4. Press the **ZERO** button. The word “TARE” will appear on the display, followed by “0.0” LB or “0.0” KG.
5. Keep the object on the platform, and then ask the patient to step onto the scale as well. The scale will automatically deduct the weight of the object and only display the patient’s weight.
6. When you remove the item from the platform, “under” will be temporally displayed and then the display will return to “0.0” LB or KG.

MAINTENANCE

Maintenance

The following pages provide instructions for maintenance, cleaning, calibrating, and troubleshooting of the 500KL scale. Maintenance operations other than those described in this manual should be performed by qualified service personnel.

Caution: Before first use, or after long periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear, and tear.
2. Inspect the AC adapter for cord cracking or fraying, or for broken/bent prongs.

Note: This scale is an extremely sensitive weighing device. If the auto-off function is disabled, battery power will be consumed.

Cleaning

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

Caution: Disconnect the scale from the AC adapter power source prior to cleaning the unit.

1. Clean all external surfaces with a clean damp cloth. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale into cleaning or other liquid solutions.
3. Do not use Isopropyl Alcohol or other solutions to clean the display surface.
4. Do not use abrasive cleaners.

Replacement Parts

Parts are available if a piece of the scale needs to be replaced. Contact Health o meter Professional Scales Customer Service at 1-800-815-6615 to inquire about the availability of these replacement parts.

Part #	Description
500HEADPIECE	Headpiece for height rod
500KLKEYPAD	Keypad overlay
500KLROD	Height rod
500KLWHEELS	Set of 2 wheels
500KLFOOT	Foot for base
500KLMAT	Replacement mat for platform
BATTDOR500	Battery compartment door
500KLPILLARFOOT	Foot for pillar
ADPT31	Power adapter

CALIBRATION

This scale has been factory calibrated, and does not require calibration prior to use. If required, the scale can be calibrated. Calibration of your 500KL scale is performed using kilograms (KG) or pounds (LB), according to the units of measure used upon entering into the calibration path. The following calibration assumes the scale is set to pounds, and 100 lbs (40 kg) / 200 lbs (80 kg) / 300 lbs (120 kg) is employed as the calibration standards.

ACTION	DISPLAY SHOWS
Press and hold the LB/KG and ZERO buttons at the same time, and press the ON/OFF button to turn on the scale. "CAL" will briefly appear on the display, followed by four numbers indicating that you can calibrate. Release all three buttons.	"CAL" followed by "XXXX"
Press the ZERO button. "C-100" (for LB mode) or "C-40" (for KG mode) will flash on the display, followed by "0" or "1".	"XXXX" followed by "C-100" (LB mode) or "C-40" (KG mode) then "0" or "1".
Place 100 lbs or 40 kg on the scale. Four numbers will appear. When calibration numbers stabilize, press the ZERO button. "C-200" (for LB mode) or "C-80" (for KG mode) will appear on the display, followed by "0" or "1".	"XXXX" followed by "C-200" (LB mode) or "C-80" (KG mode) then "0" or "1".
Place 200 lbs or 80 kg on the scale. Four numbers will appear. When calibration numbers stabilize, press the ZERO button. "C-300" (for LB mode) or "C-120" (for KG mode) will appear on the display, followed by "0" or "1".	"XXXX" followed by "C-300" (LB mode) or "C-120" (KG mode) then "0" or "1".
Place 300 lbs or 120 kg on the scale. Four numbers will appear. When calibration numbers stabilize, press the ZERO button. "END" will appear on the display, followed by "OFF". The scale will automatically shut off.	"XXXX" followed by "END" then "OFF"

TROUBLESHOOTING

Troubleshooting

Before contacting service personnel, refer to the following instructions to check and to correct any failures.

Symptom	Possible Cause	Corrective Action
Scale does not turn on	1. Dead battery 2. Faulty electrical outlet 3. Bad power supply	1. Replace batteries 2. Use a different outlet 3. Replace AC adapter
Questionable weight or the scale does not zero	1. External object interfering with the scale 2. The display did not show "0.0" before weighing 3. Scale is not placed on a level floor 4. Scale is out of calibration	1. Remove interfering object from the scale 2. Ask the patient to get off the scale, zero the scale and begin weighing process again 3. Place the scale on a level floor and begin weighing process again 4. Check weight with known weight value
Weighing is performed but the weighing process takes too long and the weight does not lock on the displayed	The patient is not standing still	Ask the patient to stand still
The display shows "Over" message	The load on the scale exceeds the capacity (500 lb / 220 kg)	Remove the excess weight and use the scale according to its limits
The display shows "Lo" message	The batteries are discharged	Replace batteries according to instructions
Scale turns on but will not register a weight reading	Load cell connector cable is not plugged in	Carefully separate the platform from the column by removing the screws in the underside of the base. Inspect the load cell connector cable to ensure it is completely plugged into the RJ jack. If cable was plugged in, unplug and plug in again until it clicks into place.

WARRANTY

Limited Warranty

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC

Attention R/A# _____

Return Department

9500 West 55th Street

McCook, IL 60525

PELSTAR, LLC

9500 West 55th Street • McCook, IL 60525 • USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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Health o meter® est une marque de commerce enregistré de Sunbeam Products, Inc. Utilisée sous licence.
Les produits Health o meter® Professional sont manufacturés, conçus, et possédés par Pelstar, LLC.